Condicionador
de Porcelanas

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0613300000 - Package containing 01 2.5ml syringe

COMPOSITION

Hydrofluoric acid at 10%, Water, Thickener, and Coloring.

INDICATIONS FOR USE

Porcelain Conditioner is a 10% hydrofluoric acid-based gel suitable for acid conditioning of porcelain with the aim to increase adhesion to the restoration.

PRECAUTIONS/WARNINGS

- For dental use only.
- Professional use.
- Keep out of the reach of children.
- High temperatures can lead to poor performance and change in consistency.
- Prolonged use can cause overconditioning of the prosthetic part, that is, over-conditioning of the prosthetic part, damaging it.
- This product contains hydrofluoric acid, avoid contact with enamel, dentin, eyes and skin. If contact occurs, wash with water saturated with sodium bicarbonate. For eyes and skin, if irritation persists, get medical attention.
- The use of PPE (personal protection equipment) is mandatory. The Dental Surgeon or assistant, during its manipulation, must wear: goggles, cap, mask, gloves and a lab coat.
- It is recommended that the Porcelain Conditioner be used with a universal disposable plastic applicator tip (commonly found on the market). After using the product, the applicator tip must be removed and discarded. The use of applicator tips, other than those recommended by the manufacturer, is the responsibility of the professional. o, a ponta aplicadora deve ser removida e descartada. A utilização de pontas aplicadoras, que não a recomendada pelo fabricante, é de responsabilidade do profissional.

STEP-BY-STEP INSTRUCTIONS

REPAIR OF PORCELAINS

- 1- Prepare a bezel of 2-3 mm, blast with aluminum oxide and wash with water spray.
- 2- Do the acid attack on the porcelain with the Porcelain Conditioner for 4-5 minutes, wash, rinse well, and dry completely.
- 3- Apply Dentsply¹ Binding Agent Silane according to the instructions for use.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

REPAIR OF PORCELAIN / METAL

- 1- Prepare the porcelain (bezel of 2-3mm).
- 2- Blast the metal and the porcelain with aluminum oxide, wash with water spray and dry.
- 3- Apply the Porcelain Conditioner only on the porcelain surface for 4-5 minutes, wash, rinse, and dry.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

REPAIR OF HYBRID COMPOSITES

- 1- Prepare a bezel of 2-3mm, blast with aluminum oxide and wash with water.
- 2- Apply the Porcelain Conditioner over the restoration, for 60 seconds, wash with water spray, and dry completely.
- 3- Apply Dentsply¹ Bonding Agent Silane according to the instructions.
- 4- Apply the adhesive and the composite according to the instructions from the manufacturer.

PREPARATION OF PORCELAIN VENEER FACETS

- 1- Isolate the external face of the porcelain veneer facets with wax.
- 2- Apply the Porcelain Conditioner on the internal face of the porcelain veneer facet for 4-5 minutes, wash carefully with water spray or ultrasonic bath and dry.
- 3- Apply Dentsply¹ Bonding Agent Silane according to the instructions.
- 4- Cementation with EnForce Fluorine[®] resin cement, as instructed for adhesive cementation of porcelain veneer veneers.

¹Products sold separately

CARE FOR STORAGE

Keep away from heat and humidity.

HANDLING CONDITIONS

There are no special handling conditions. Dispose of the product according to local legislation.

ADVERSE EFFECTS:

Allergic response or burn if inadvertently comes into contact with the patient.

SINGLE USE PRODUCT. PROHIBITED REPROCESSING

LOT Nº

See Label.

VALIDITY

2 years.

ATTENTION: Check the correlation of the version of these instructions for use with the product purchased indicated on the package.

To obtain these instructions for use free of charge in printed format, request our service by calling 0800 771 2226 (Brazil only) or by e-mail atendimento@dentsplysirona.com