UK/CA/AU

PRODUCT DESCRIPTION

Single use hydrophilic urinary catheter.

MATERIAL

Hydrophilic coated urinary catheter. Polyolefin-based elastomer (POBE) with PVP and NaCl coating.

INDICATIONS FOR USE

For short and long term intermittent urinary catheterization.

PATIENT POPULATION

For adults, adolescents, children, and infants.

WARNINGS AND PRECAUTIONS

For single use only.

Once used the surface coating will deteriorate and no longer sterile.

Reuse may lead to discomfort,

urethral damage, infection.

Do not use a product if the sterile packaging is broken or damaged.

Urinary catheterization therapy is associated with an increased risk of urethral bleeding, trauma and/or infection.

LoFric catheters are for prescription use only.

AU: Always consult a healthcare professional before using LoFric catheters.

Follow instructions and advice from your healthcare professional.

Contact your prescriber if you experience difficulties.

ADVERSE REACTIONS

Common adverse reactions (>1/100) related to catheterization therapy includes urethral damage and urinary tract infection.

If unexpected discomfort, sign of trauma or infection occurs, discontinue use and consult your prescriber.

Any serious adverse reaction occurring when using the catheter should be reported to the manufacturer and your local health authority.

PACKAGE AND STORAGE

Store in their package in a dry place, at room temperature.

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