Whitegold
home

Mantém fora do alcance de crianças
Mantener fuera del alcance de los niños
Keep out of reach of children

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WHITEGOLD HOME

Dental Whitening System

Whitegold Home is a highly recommended system whitening of vital teeth. Whitegold Home contains carbamide peroxide as active substance, which has shown to be an effective and safe vital teeth whitening agent, in addition to containing fluorine in its formulation, functioning as de-sensitizing agent and pleasant mint flavor.

PRESENTATIONS

15131510000 - WHITEGOLD HOME 16%, pack containing 1 syringe of 3 ml
 15130530000 - WHITEGOLD HOME 16%, packaging containing 2 syringes of 3 ml, 2 silicone plates, 1 tray
 15130540000 - WHITEGOLD HOME 16%, pack containing 3 syringes of 3 ml
 15131530000 - WHITEGOLD HOME 16%, packaging containing 5 syringes of 3 ml, 2 silicone plates, 1 tray
 15131580000 - WHITEGOLD HOME 16%, package containing 8 syringes of 3 ml, 2 silicone plates, 1 tray.

COMPOSITION

Sodium Benzoate, Propylene glycol, EDTA, Triethanolamine, Carbamide Peroxide, Sodium fluoride, Mint Flavor, Menthol, Carbopol 934.

INDICATIONS

White Gold Home is indicated for whitening vital teeth with chronic intrinsic color changes, such as, for example, natural darkening (naturally yellowish or grayish teeth); physiological darkening.

Straining provoked by the excessive ingestion of drugs (tetracycline and fluorides) *.

As a complementary whitening treatment associated to the office treatment with Whitegold Office (mixed technique).

As an auxiliary method in the whitening treatment of teeth strained due to endodontic treatment.

Dental whitening prior to restoring treatment and prosthetic procedures.

*The more severe chromatic alterations due to excessive use of drugs, such as, for example, tetracycline and fluorides, are more resistant and difficult to treat, and procedures association may be required, as whitening procedures associated to restoring procedures.

CONTRAINDICATIONS

- Patients allergic to hydrogen peroxide or other products in the formulation.

PRECAUTIONS, WARNINGS AND SPECIAL CARE

Avoid contact of the whitening gel with the oral tissues, eyes and skin. If accidental contact occurs, wash the affected area with a generous amount of water.

In case of contact with eyes, rinse immediately with plenty of water and seek medical attention. Keep out of the reach of children. White Gold Home is a product based on Carbamide Peroxide and is considered an inactive peroxide and maintained as such if stored at temperatures between 5 to 30°C. The storage of Carbamide Peroxide-based lighteners at temperatures higher than those recommended may eventually start the transformation of the product to its active form (Hydrogen Peroxide) before coming into contact with the oral cavity, without, however, bringing any risk to the patient. On exposure to higher temperatures for prolonged periods, this transformation becomes unwanted, increasing the fluidity of the product and may interfere with the final result.

USE INSTRUCTIONS**A) PREPARATION OF THE MOUTH TRAY FOR WHITENING**

- Carefully, make a mouth tray of the dental arches to be whitened using alginate (Jeltrate Plus or Jeltrate Orthodontic Chromatic, for example).
- Build the plaster model using plaster Durone IV. For the upper model, it is not necessary to reproduce the palate region.
- Cut the plaster models in a way that the their bases are parallel to the occlusal plane of the teeth.
- With the totally dry models, in case opting to prepare the relief apply a fluid photo-active resin to create a space of approximately 0.5 to 1.0 mm on the vestibular face of the teeth. Take in mind that this relief shall not be performed in the incise borders, occlusive surfaces or inter-proximal. In case not opting to prepare it, proceed immediately with the subsequent stage.
- Fix the plate for the mouth tray on the plastifying device, plastify it and make the tray. OBS: It is recommended to use silicone plates, with thickness of approximately 1.0mm.
- Cut the mouth tray in a way to be 0.5mm beyond the gingival borders. The rounding of the edges can be done with siliconized drills, as for example, Enhance.

B) APPLICATION OF THE WHITENING GEL

Before starting the whitening treatment, it is recommended to totally remove the calculus and/or perform dental prophylaxis with the purpose to eliminate calculus, bacterial plaque and extrinsic pigments present on tooth surfaces. The whitening degree will depend on some factors, among them:

- Time of daily use of the mouth tray with the gel.
- Number of days using the mouth tray.
- Susceptibility of the patient's individual: capacity of peroxide diffusion determined by the higher or the lower permeability of the dental structures.
- The patient's cooperation in the whitening treatment - related to the time of use.

NIGHT WHITENING PROGRAM

- Before going to bed, brush the teeth and pass dental floss.
- Apply a small quantity of Whitegold Home whitening gel inside the mouth tray. Take care not to overload the tray.
- Position the mouth tray on the teeth and remove the excess of whitening gel with the help a tooth brush.
- Use the mouth tray for 4 to 8h at night, it is important to point out that the period with the highest efficiency of the gel is up to 4 hours. In the morning, remove the tray; brush the teeth, removing all whitening gel remnants.
- Wash the mouth tray completely and keep it in its proper casing.

DAY WHITENING PROGRAM

- Brush the teeth and pass dental floss.
- Pour small quantity of Whitegold Home inside the mouth tray. Do not overload the tray.
- Position the mouth tray on the teeth and remove the excess of whitening gel with the help a tooth brush.
- Use the mouth tray for at least 2 hours.
- Do not eat, drink or smoke while you are using the home whitening mouth tray.
- Remove the mouth tray; brush the teeth to remove all whitening gel remnants.
- Wash the mouth tray completely and keep it in its proper casing.

STORAGE

Store Whitegold Home protected from direct sun light at temperatures between 5° and 30°C in order to maintain the product stability and guarantee its maximum efficiency.

VALIDITY PERIOD

24 months.

PROHIBITED REPROCESSING SINGLE USE USE UNDER PROFESSIONAL GUIDANCE

ATTENTION: Check the correlation of the version of these instructions for use with the product purchased indicated on the package. To obtain these instructions for use free of charge in printed format, request our service by calling 0800 771 2226 (Brazil only) or by e-mail [customer atendimento@dentsplysirona.com](mailto:atendimento@dentsplysirona.com)