RECIPROC® Blue

EN	Instructions For Use	2
FR	Instructions d'utilisation	9
IT	Istruzioni per l'uso	16
DE	Gebrauchsanweisung	23
ES	Instrucciones de uso	30
PT	Instruções de utilização	37
NL	Gebruiksaanwijzing	44
SV	Användarinstruktioner	51
DA	Brugsanvisning	58
NO	Brukerinstruksjoner	65
FI	Käyttöohjeet	72
PL	Instrukcja obsługi	79
LT	Naudojimo instrukcija	86
LV	Lietošanas norādījumi	93
ET	Kasutusjuhised	100
CS	Návod k použití	107
SK	Návod na použitie	114
HU	Használati utasítás	121
RO	Instrucțiuni de utilizare	128
SR	Uputstva za upotrebu	135
HR	Upute za uporabu	142
BG	Инструкции за употреба	149
BS	Uputstva za upotrebu	156
MK	Упатство за употреба	163
EL	Οδηγίες χρήσης	170
UK	Інструкції із застосування	177
TR	Kullanım Kılavuzu	184
KO	사용 지침	191
ID	Petunjuk Penggunaan	198
PT-BR	Instruções de uso	205

RECIPROC® Blue

Endodontic instruments

Instructions
For Use

A brand of
 Dentsply
Sirona

FOR DENTAL USE ONLY
FOR PROFESSIONAL USE ONLY
SINGLE USE – REPROCESSING PROHIBITED
STERILE – STERILIZED BY RADIATION

INSTRUCTIONS FOR USE

SINGLE USE STERILE ENDODONTIC INSTRUMENTS

REF. V040252, V040052

1 INDICATIONS FOR USE

- Indication for use:
The products are indicated for treatment of endodontic disease.
- Intended purpose:
Engine-driven instrument intended for root canal preparation and retreatment (shaping and debridement of the root canal).
- Intended user:
Endodontic instruments are to be used only in a clinical or hospital environment, following good dental practice, by qualified dental professionals such as general practitioners as well as endo specialists (endodontist) and dental assistants.
- Instruments shall be used in combination with an endodontic motor (manually in severe curvatures).

2 CONTRAINDICATIONS

None.

3 CONTENT, COMPOSITION AND COMPATIBLE DEVICES

RECIPROC® Blue instruments are made of three main components: the working part made of a nickel-titanium alloy with a heat treatment, a colored silicone stopper and a plated brass shank with an ABS plastic ring.

- The files are manufactured with a process that results in a file that has a colored appearance. The files may appear slightly curved due to this process. This is not a manufacturing defect, and it is not necessary to straighten the file prior to use. Once inside the canal, the file will follow the root canal anatomy.
- Use in a reciprocating motion with a designated drive system having the approved RECIPROC® settings.
- Lubricants such as NaOCl, EDTA, ProLube, Glyde™ shall be used.
- The use of radiographs in combination with an apex locator and a tool for adjusting the silicone stopper to the correct working length is the appropriate method of working length determination.

4 WARNINGS

- Strictly follow the Instructions For Use to minimize the following risks to the devices, the patient and/or the user:
 - Breakage of instrument.
 - Cross-contamination.
 - Heat generation due to insufficient lubrication and irrigation.
 - Swallowing of working part of the instrument.
 - Toxic or allergic reactions caused by processing residues.
- Similar to all machine-driven root canal instruments, RECIPROC® Blue instruments should not be used in a root canal with an abrupt apical curvature due to heightened risk of separation. In this case, pre-curved hand files should be used in the apical region.

5 PRECAUTIONS

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Use a rubber dam system during the endodontic procedure.
- For your own safety, wear personal protective equipment (gloves, glasses, mask).
- Inspect the packaging before use and do not use the instruments if the packaging is damaged.
- Do not use the instruments after expiration date.
- Check the instrument before each use for signs of defects such as deformations (bent, unwound), breakage, corrosion, damaged cutting edges, loss of color coding or marking. With these indications the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Before using any instrument, make sure it is well connected to the contra-angle head.
- Check instrument and clean working part frequently during instrumentation, inspecting for signs of distortion, elongation or wear, such as uneven flutes, dull spots. With these indications that the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Instruments should not be fully immersed in a sodium hypochlorite solution (NaOCl). Only the working part of the NiTi instrument, which is in contact with the patient should be immersed in a NaOCl solution concentrate at NOT more than 5%.
- Exercise caution in the apical area and in canals that divide, and/or exhibit abrupt curvatures or recurvatures.
- Irrigate abundantly and frequently the canal throughout the procedure and after each instrument used (according to good dental practices).
- Always use minimal apical pressure. Never force the files down the canal.
- When instrument does not easily progress, clean and inspect the cutting flutes, then irrigate, recapitulate with a manual file and reirrigate.

- For shaping extremely curved canals it is safer to use the file only to shape one canal in order to reduce the risk of breakage. Pay attention to the following good practices:
 - Use a new file and discard it after the canal was treated (single canal use).
 - Use manual instead of rotary files.
 - Use small size, flexible and/or NiTi files (this will help avoid canal transportation).
 - Visually inspect the working part for all the defects listed in the former paragraph during use (i.e. after each wave).
 - Avoid the standard reaming continual rotational motion and instead use small angle motions (filing motion, watch winding oscillation motion, or balanced force technique) in order to limit the rotational bending fatigue on the instruments and improve their expected life.

6 ADVERSE REACTIONS / UNDESIRABLE SIDE EFFECTS

Temporary post-operative discomfort could arise generally as a result of receiving a root canal treatment.

7 STORAGE CONDITIONS

Keep products in a dry and clean environment, away from sources of moisture and direct sunlight.

8 STEP BY STEP INSTRUCTIONS

Shaping procedure with RECIPROC® Blue

Select the correct RECIPROC® Blue instrument:

In most cases, size R25 will be suitable for the root canal treatment.

- 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
 - a) If the canal is partially or completely invisible on the radiograph: The canal is considered narrow = use size R25.
 - b) If the canal is completely visible on the radiograph: Take an ISO size 30 hand instrument and insert it passively into the canal:
 - If it reaches working length, the canal is considered large = use size R50.
 - If an ISO size 30 hand instrument does not go passively to working length, take an ISO size 20 hand instrument. If file goes passively to length, the canal is considered medium = use size R40.
 - If an ISO size 20 hand instrument does not go passively to length, the canal is considered narrow = use size R25.

Note: Passively means that the instrument goes directly to working length without any watch-winding and can be moved up-and-down without shaping action. The amplitude of the up-and-down movement should be 4-5 mm.

- 2) Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.

- 3) Irrigate abundantly and frequently after each instrument used (according to good dental practices).
- 4) Starting preparation depending on the instrument size:
 - a) Size R25: Prior to preparation with an R25, estimate the working length with the help of a pre-operative radiograph. Set the stopper at 2/3 of that length.
 - b) Sizes R40 and R50: Before selecting the appropriate RECIPROC® Blue instrument, check the working length electronically by using a C-PILOT® or K-File. Set the stopper at that length.
- 5) With irrigant in the canal, introduce the RECIPROC® Blue instrument into the canal. Press the ON/OFF button or foot pedal of the motor when the instrument is at the root canal orifice.
- 6) Move the instrument in a slow in-and-out pecking motion until approx. 2/3 of the working length has been reached (indicated by the silicone stopper). One in-and-out movement = one peck. Clean the instrument after each pecking motion. The amplitude of the in-and-out movements should not exceed 3 mm. Only very light pressure should be applied. The instrument will advance easily in the canal.
- 7) If necessary, the instrument can be used in a brushing motion to enlarge the root canal entrance.
- 8) After each pecking motion, make sure the canal is free with an ISO size 010 C-PILOT® file.
- 9) Reconfirm the working length, before continuing shaping until full working length has been reached.
- 10) Gauge the foramen with a hand instrument which is one ISO size larger than the RECIPROC® Blue instrument that reached full working length.
 - a) If the instrument is snug at length, preparation is finished and the canal is ready to be obturated.
 - b) If the instrument is loose at length, use next size of RECIPROC® Blue instrument, and, when necessary proceed with the next file size, gauging after each instrument respectively with a hand files of the same size. Use the same shaping technique up to working length.
- 11) Use dedicated paper points to dry and dedicated gutta-percha points to obturate the root canals.

Retreatment procedure with RECIPROC® Blue

- 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
- 2) Remove the bulk of the gutta-percha in the coronal third of the canal with an appropriate instrument e.g. Gates Enlarger or with the help of an ultrasonic device.
- 3) If necessary, soften gutta-percha with a solvent (e.g. eucalyptus oil).
- 4) Use size R25 as described until working length has been reached. If resistance is encountered, do not apply pressure.
- 5) Remove the instrument from the canal, apply a little solvent and repeat the process. The carrier may be removed in one piece during the use of the RECIPROC® Blue instrument; otherwise, it will be removed in small pieces with the gutta-percha.
- 6) Use a brushing motion against lateral walls to remove residual obturation material.

- 7) After reaching working length with size R25, use size R40 or R50 for an increased apical enlargement if mandatory.

9 HYGIENE, DISINFECTION, CLEANING AND STERILIZATION

- RECIPROC® Blue instruments are single use devices and cannot be reused. Re-use can increase the risk of cross contamination or breakage.
- Sterility cannot be guaranteed if packaging is open, damaged or wet.
- Disinfection, cleaning and sterilization are not applicable.
- Products shall be disposed according to local regulations for the safe disposal of sharp and contaminated devices.

10 ADDITIONAL INFORMATION

- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
- See webpage dentsplysirona.com/ifu to:
 - Get a free print copy of the IFU.
 - Get an explanation of non-harmonized symbols for IFUs and labels.

DENTSPLY DeTrey GmbH
De-Trey-Straße 1
78467 Konstanz
Germany

Manufacturer

Maillefer Instruments Holding Sàrl
Chemin du Verger 3
1338 Ballaigues
Switzerland

dentsplysirona.com/ifu

EU Importer

Depending on the country of distribution, the devices may be imported by one of the following importers¹:

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Importer 2

VDW GmbH
Bayerwaldstr. 15
81737 München
Germany

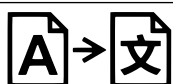
¹For further information regarding the actual importer of the devices, please contact:
ds-distribution-dse-quality@dentsplysirona.com

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