R-PILOT™

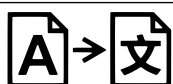
EN	Instructions For Use	2
FR	Instructions d'utilisation	8
IT	Istruzioni per l'uso	14
DE	Gebrauchsanweisung	20
ES	Instrucciones de uso	26
PT	Instruções de utilização	32
NL	Gebruiksaanwijzing	38
SV	Användarinstruktioner	44
DA	Brugsanvisning	50
NO	Brukerinstruksjoner	56
FI	Käyttöohjeet	62
PL	Instrukcja obsługi	68
LT	Naudojimo instrukcija	74
LV	Lietošanas norādījumi	80
ET	Kasutusjuhised	86
CS	Návod k použití	92
SK	Návod na použitie	98
HU	Használati utasítás	104
RO	Instrucțiuni de utilizare	110
SR	Uputstva za upotrebu	116
HR	Upute za uporabu	122
BG	Инструкции за употреба	128
BS	Uputstva za upotrebu	134
MK	Упатство за употреба	140
EL	Οδηγίες χρήσης	146
UK	Інструкції із застосування	152
TR	Kullanım Kılavuzu	158
KO	사용 지침	164
ID	Petunjuk Penggunaan	170
PT-BR	Instruções de uso	176

R-PILOT™

Endodontic instruments

Instructions
For Use

A brand of
 Dentsply
Sirona

FOR DENTAL USE ONLY
FOR PROFESSIONAL USE ONLY
SINGLE USE – REPROCESSING PROHIBITED
STERILE – STERILIZED BY RADIATION

INSTRUCTIONS FOR USE

SINGLE USE STERILE ENDODONTIC INSTRUMENTS

REF. V040217, V041217

1 INDICATIONS FOR USE

- Indication for use:
The products are indicated for treatment of endodontic disease.
- Intended purpose:
Engine-driven instrument intended for root canal preparation (glide path).
- Intended user:
Endodontic instruments are to be used only in a clinical or hospital environment, following good dental practice, by qualified dental professionals such as general practitioners as well as endo specialists (endodontist) and dental assistants.
- Instruments shall be used in combination with an endodontic motor (manually in severe curvatures).

2 CONTRAINDICATIONS

None.

3 CONTENT, COMPOSITION AND COMPATIBLE DEVICES

R-PILOT™ instruments are made of three main components: the working part made of a nickel-titanium alloy (M-Wire®), a colored silicone stopper and a plated brass shank with an ABS plastic ring.

- Use in a reciprocating motion with a designated drive system having the approved RECIPROC® settings.
- Lubricants such as NaOCl, EDTA, ProLube, Glyde™ shall be used.
- The use of radiographs in combination with an apex locator and a tool for adjusting the silicone stopper to the correct working length is the appropriate method of working length determination.

4 WARNINGS

- Strictly follow the Instructions For Use to minimize the following risks to the devices, the patient and/or the user:
 - Breakage of instrument.
 - Cross-contamination.

- Heat generation due to insufficient lubrication and irrigation.
- Swallowing of working part of the instrument.
- Toxic or allergic reactions caused by processing residues.
- Similar to all machine-driven root canal instruments, R-PILOT™ instruments should not be used in a root canal with an abrupt apical curvature due to heightened risk of separation. In this case, pre-curved hand files should be used in the apical region.

5 PRECAUTIONS

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Use a rubber dam system during the endodontic procedure.
- For your own safety, wear personal protective equipment (gloves, glasses, mask).
- Inspect the packaging before use and do not use the instruments if the packaging is damaged.
- Do not use the instruments after expiration date.
- Check the instrument before each use for signs of defects such as deformations (bent, unwound), breakage, corrosion, damaged cutting edges, loss of color coding or marking. With these indications the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Before using any instrument, make sure it is well connected to the contra-angle head.
- Check instrument and clean working part frequently during instrumentation, inspecting for signs of distortion, elongation or wear, such as uneven flutes, dull spots. With these indications that the devices are not able to fulfil the intended use with the required safety level, instruments should be discarded.
- Instruments should not be fully immersed in a sodium hypochlorite solution (NaOCl). Only the working part of the NiTi instrument, which is in contact with the patient should be immersed in a NaOCl solution concentrate at NOT more than 5%.
- Exercise caution in the apical area and in canals that divide, and/or exhibit abrupt curvatures or recurvatures.
- Irrigate abundantly and frequently the canal throughout the procedure and after each instrument used (according to good dental practices).
- Always use minimal apical pressure. Never force the files down the canal.
- When instrument does not easily progress, clean and inspect the cutting flutes, then irrigate, recapitulate with a manual file and reirrigate.
- For shaping extremely curved canals it is safer to use the file only to shape one canal in order to reduce the risk of breakage. Pay attention to the following good practices:
 - Use a new file and discard it after the canal was treated (single canal use).
 - Use manual instead of rotary files.
 - Use small size, flexible and/or NiTi files (this will help avoid canal transportation).

- Visually inspect the working part for all the defects listed in the former paragraph during use (i.e. after each wave).
- Avoid the standard reaming continual rotational motion and instead use small angle motions (filing motion, watch winding oscillation motion, or balanced force technique) in order to limit the rotational bending fatigue on the instruments and improve their expected life.

6 ADVERSE REACTIONS / UNDESIRABLE SIDE EFFECTS

Temporary post-operative discomfort could arise generally as a result of receiving a root canal treatment.

7 STORAGE CONDITIONS

Keep products in a dry and clean environment, away from sources of moisture and direct sunlight.

8 STEP BY STEP INSTRUCTIONS

The application of R-PILOT™ is recommended for the following cases:

- **Initially** to render the existing natural canal path smooth prior to the use of shaping instruments.
- **During root canal preparation with RECIPROC®** whenever advancement with RECIPROC® becomes difficult (e.g. RECIPROC® R25 or RECIPROC® Blue R25).
 - 1) Review different horizontally angulated radiographs to diagnostically determine the width, length, and curvature of any given root canal.
 - 2) Create straight-line access to the canal orifice(s) with emphasis on flaring, flattening, and finishing the internal axial walls.
 - 3) Irrigate abundantly and frequently after each instrument used (according to good dental practices).
 - 4) Scout the root canal with hand files like C-PILOT®, at least an ISO 008, determine the working length by using an electronic apex locator in combination with radiographs, irrigate and confirm patency.
 - 5) With irrigant in the canal, introduce the R-PILOT™ instrument into the canal. Press the ON/OFF button or foot pedal of the motor when the instrument is at the root canal orifice.
 - 6) Move the instrument in a slow in-and-out pecking motion until the full working length has been reached. One in-and-out movement = one peck. Clean the instrument after each pecking motion. The amplitude of the in-and-out movements should not exceed 3 mm. Only very light pressure should be applied. It is also possible to brush laterally. The instrument will advance easily in the canal.
 - 7) Irrigate the expanded glide path.
 - 8) Reconfirm the working length, before shaping with the VDW file system of your choice.

9 HYGIENE, DISINFECTION, CLEANING AND STERILIZATION

- R-PILOT™ instruments are single use devices and cannot be reused. Re-use can increase the risk of cross contamination or breakage.
- Sterility cannot be guaranteed if packaging is open, damaged or wet.
- Disinfection, cleaning and sterilization are not applicable.
- Products shall be disposed according to local regulations for the safe disposal of sharp and contaminated devices.

10 ADDITIONAL INFORMATION

- Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
- See webpage dentsplysirona.com/ifu to:
 - Get a free print copy of the IFU.
 - Get an explanation of non-harmonized symbols for IFUs and labels.

DENTSPLY DeTrey GmbH
De-Trey-Straße 1
78467 Konstanz
Germany

Manufacturer

Maillefer Instruments Holding Sàrl
Chemin du Verger 3
1338 Ballaigues
Switzerland

dentsplysirona.com/ifu

EU Importer

Depending on the country of distribution, the devices may be imported by one of the following importers¹:

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Importer 2

VDW GmbH
Bayerwaldstr. 15
81737 München
Germany

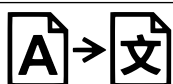
¹For further information regarding the actual importer of the devices, please contact:
ds-distribution-dse-quality@dentsplysirona.com

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